



The RELJA Clamp™

A *Clear* and *Simple* reproducible technique.

The RELJA Clamp is a sterile, single-use clamp packaged for surgical correction of first metatarsal deformities across three planes. Utilizing a no-incision technique, it features a radiolucent design enabling correction assessment before fixation.

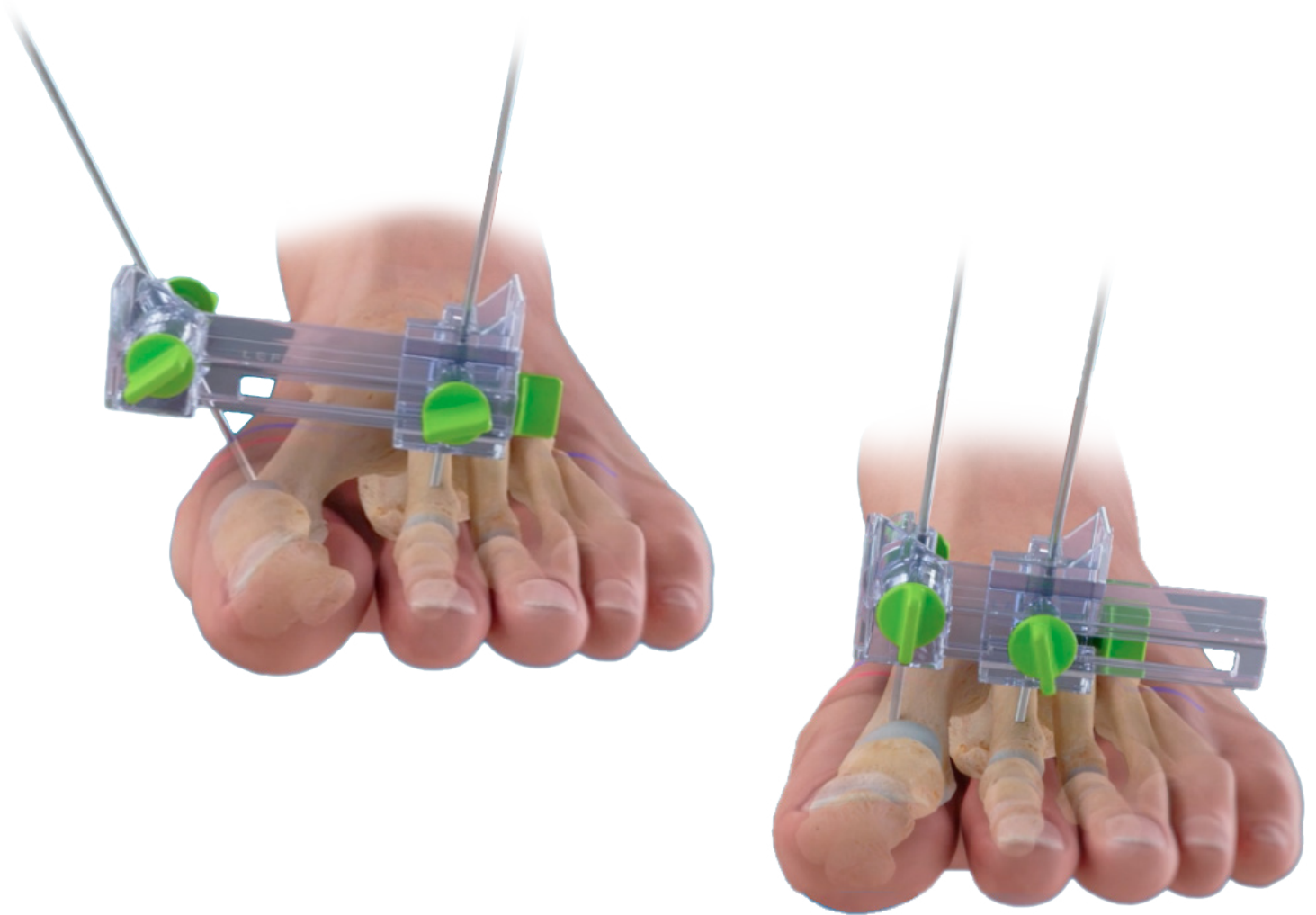


Perform standard joint preparation of the first TMT before applying the RELJA Clamp.

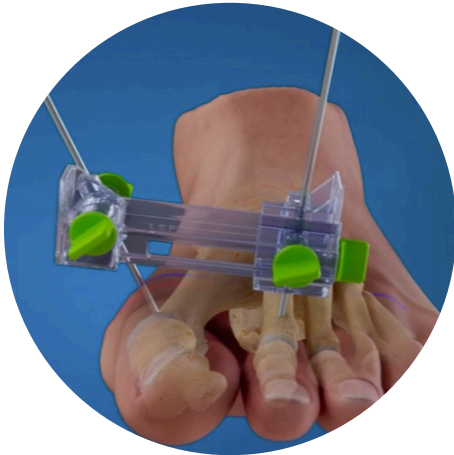


Step One

Fixate the clamp with the provided Steinmann pins into the first and second metatarsals.

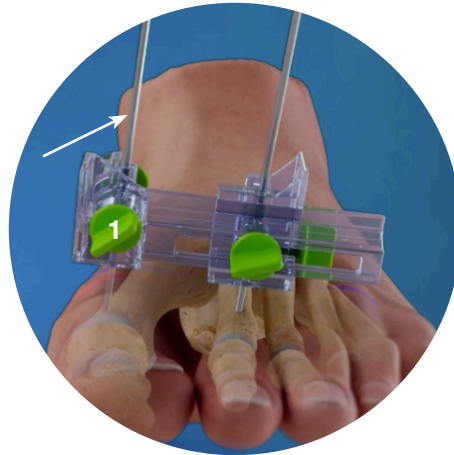


Step Two

Rotate the first metatarsal with the first pin until proper frontal correction is achieved then tighten the first thumb screw to lock into place.

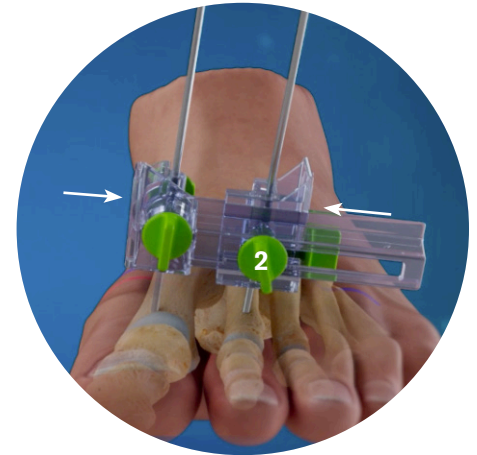


Step Three

Compress the tabs together to reduce the first intermetatarsal angle then tighten the second thumb screw to lock into place.

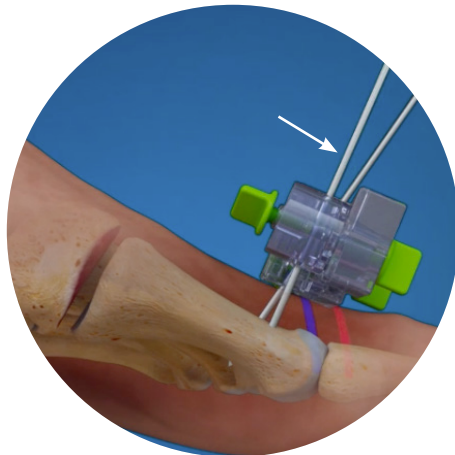

Step Four

Plantarflex the first metatarsal into the desired position by pressing the first pin down.

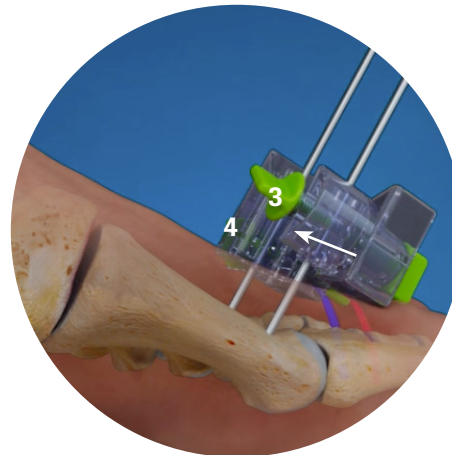

Step Five

Compress the first TMT joint and tighten the third and fourth thumb screws to lock into place. Fixate the joint with preferred fixation method.

Item #	Description
RI-CC-LR	RELJA CLAMP

The RELJA Clamp is a product of RELJA Innovations. For more information visit www.Reljainnovations.com

This content is provided as an educational tool only and is not meant as medical advice in the usage of specific BioPro products. A healthcare professional must use their professional judgment in making any final determinations in product usage and technique. The product's Instructions for Use, should always be reviewed prior to surgery. Postoperative management is patient-specific and dependent on the treating professional's assessment. Individual results will vary and not all patients will experience the same postoperative activity level or outcomes. This information does not constitute medical, legal, or any other type of professional advice and should not be relied upon as such. It is not to be redistributed, duplicated, or disclosed without the express written consent of BioPro, Inc.



BioPro, Inc.
2929 Lapeer Road, Port Huron, MI 48060, USA
info@bioproimplants.com | 1-810-982-7777
www.bioproimplants.com



intertek